

WITNESS® | FFH

FELINE HEARTWORM-FELINE LEUKEMIA VIRUS ANTIGEN-
FELINE IMMUNODEFICIENCY VIRUS ANTIBODY TEST KIT



TRIPLE THREAT

DETECT THREE CRITICAL DISEASES
WITH ONE SIMPLE, CONVENIENT, AND
COST-EFFECTIVE TEST

WITNESS® FFH is a three-well test kit that detects
antigens to feline leukemia (FeLV) and feline heartworm
and antibodies to feline immunodeficiency virus (FIV).

Why choose WITNESS FFH?

WITNESS RELIABILITY

WITNESS FFH provides triple test performance you can trust.

WITNESS VALUE

WITNESS FFH is as affordable as most dual FeLV-FIV tests
on the market with the added benefit of offering heartworm
testing. Essentially, you and your patients will be getting
three tests for the price of two!

WITNESS SIMPLICITY

WITNESS FFH is easy to use and requires less prep time, fewer
steps, and no waiting time.

DETECT. PREVENT. TREAT.

zoetis

CONFIDENTLY TAKE ON THE TRIPLE THREAT OF FeLV, FIV, AND HEARTWORM.

WITNESS® FFH VERSUS IDEXX SNAP® FELINE TRIPLE TEST*

| | WITNESS FFH | | IDEXX SNAP FELINE TEST |
|----------------|--|-----|---|
| STORAGE | No refrigeration required. | vs. | Refrigeration required. |
| PREP | None required. | vs. | Allow time for components to warm to room temperature before use. Mixing of sample required before it can be added to the test. |
| TESTING | Add buffer and get results in approximately 10 minutes. | vs. | Requires time-sensitive manipulation (snapping) before waiting for results. |
| RESULTS | Simple-to-interpret results: one line for negative and two lines for positive. | vs. | Results are shown in a series of dots that may be more difficult to interpret. |

DATA FROM STUDIES COMPARING PERFORMANCES OF WITNESS® FFH VS. IDEXX SNAP® FELINE TRIPLE^{1,2*}

| FeLV | Sensitivity* | Specificity† |
|---------------------------|--------------------------|--------------------------|
| WITNESS FFH | 95.9% (90.6% - 98.6%) | 98.0% (93.8% - 99.6%) |
| SNAP Feline Triple | 95.9% (90.6% - 98.6%) | 100.0% (97.6% - 100%) |

| FIV | Sensitivity | Specificity |
|---------------------------|--------------------------|--------------------------|
| WITNESS FFH | 100% (97.2% - 100%) | 95.8% (91.0% - 98.4%) |
| SNAP Feline Triple | 93.2% (86.5% - 97.1%) | 95.0% (90.0% - 97.9%) |

| Heartworm | Sensitivity | Specificity |
|---------------------------|--------------------------|--------------------------|
| WITNESS FFH | 91.4% (83.8% - 96.1%) | 99.2% (96.3% - 99.9%) |
| SNAP Feline Triple | 93.8% (87.0% - 97.6%) | 98.4% (95.0% - 99.7%) |

*Sensitivity represents the ability to correctly identify positive samples.

†Specificity represents the ability to correctly identify negative samples.

*Based on manufacturer supplied direction insert.

TRUSTED CUSTOMER SUPPORT

For more information, visit www.zoetisus.com/simplysmarterchoice.

To order, call Zoetis customer service at 1-888-ZOETIS-1 (963-8471) or visit shop.zoetisus.com.

References:

1. Data on file, Study Report No. D880W-US-17-042, Zoetis Inc.
2. Data on file, Study Report No. D886Z-US-17-043, Zoetis Inc.

IDEXX SNAP® is a registered trademark of IDEXX Laboratories, Inc.

All trademarks are the property of Zoetis Services LLC or a related company or a licensor unless otherwise noted.

©2017 Zoetis Services LLC. All rights reserved. WIT-00257

DETECT. PREVENT. TREAT.

zoetis